



About Us

Island Polymer Industries GmbH is a leader in Cellulose Triacetate Film Production. Headquartered in Wolfen, Germany, IPI GmbH produces high quality films that are sold globally to a wide spectrum of industries, including the LCD, Optical Lens, and Photographic Film markets, as well as for Technical and Medical Applications.

Our production facility is 35,000 m² and is located in high-tech Chemiepark, Bitterfeld-Wolfen, Germany.

Contact:

IGE Medical Devices LLC
267 E. Jericho Turnpike
Mineola, NY 11501
+1 516 746 2100

David LaMaster
David@islandgroup.com

Manufactured By:

Island Polymer Industries GmbH
ChemiePark Bitterfeld- Wolfen
Area A - Andresenstraße 6, Geb. 291
D- 06766 Bitterfeld-Wolfen

www.islandpolymer.com

IPI Pathocut[®]

Manual Coverslipper

IGE MEDICAL
DEVICES



***The first
manual
coverslipper***


Your first choice for manual
coverslipping of
Clinical Pathology, Histology
and Cytology specimens

IPI Pathocut[®]

- No mounting media required
- Protected storage of the coverslipping film
- ISO compliant standardizable procedure
- Activation with only two drops of Xylene
- Immediately usable for further examination
- Repair slides safely, quickly and easily
- Unit provides precision coverslipping

IPI Pathotape[®]

IPI Pathotape[®] coverslipping film offers an innovative and efficient way to handle Histology, Clinical Pathology, Cytology, Microbiology and Hematology slides.

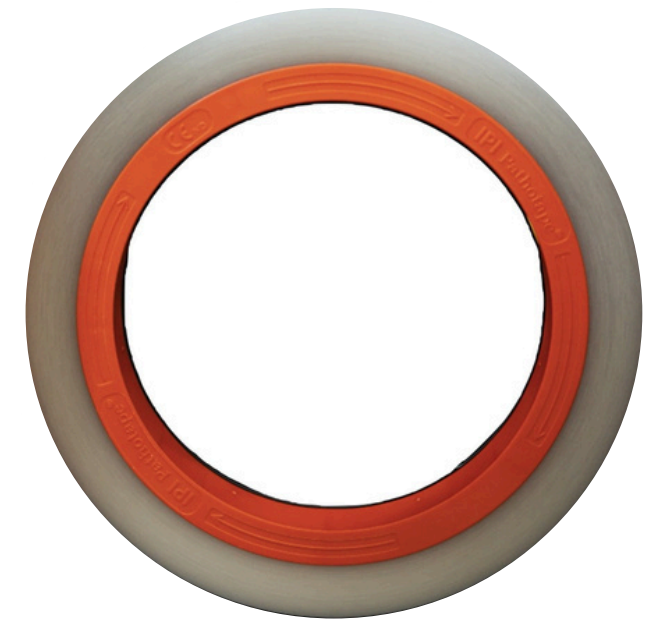

1



Place the glass slide

2




Pick up the Xylene with pipette

3




Apply two drops

4

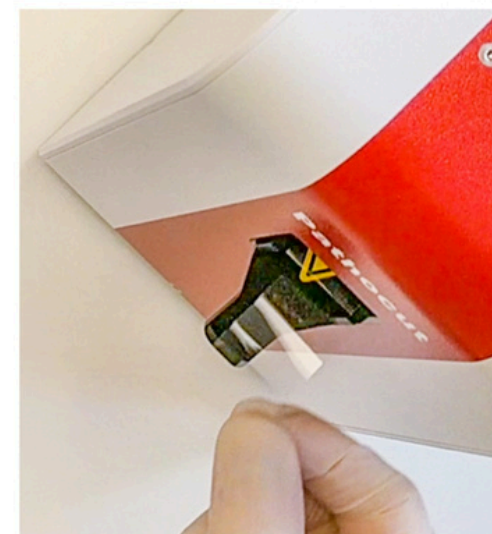

Immediately turn the crank and pick up the coverslip

5

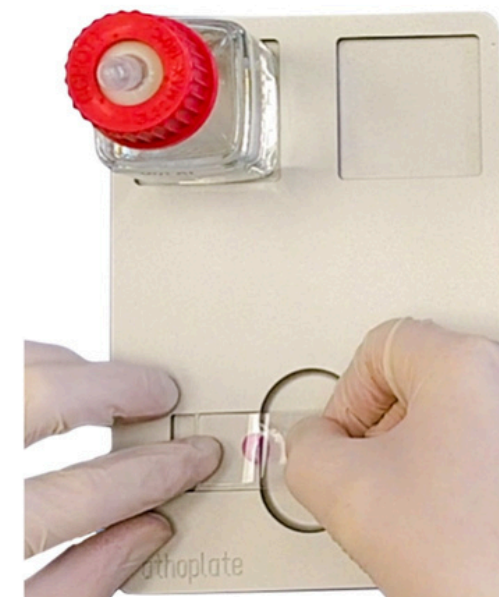

Place the coverslip on the glass slide

6



Remove the finished glass slide